


CELLER⁸

USER MANUAL

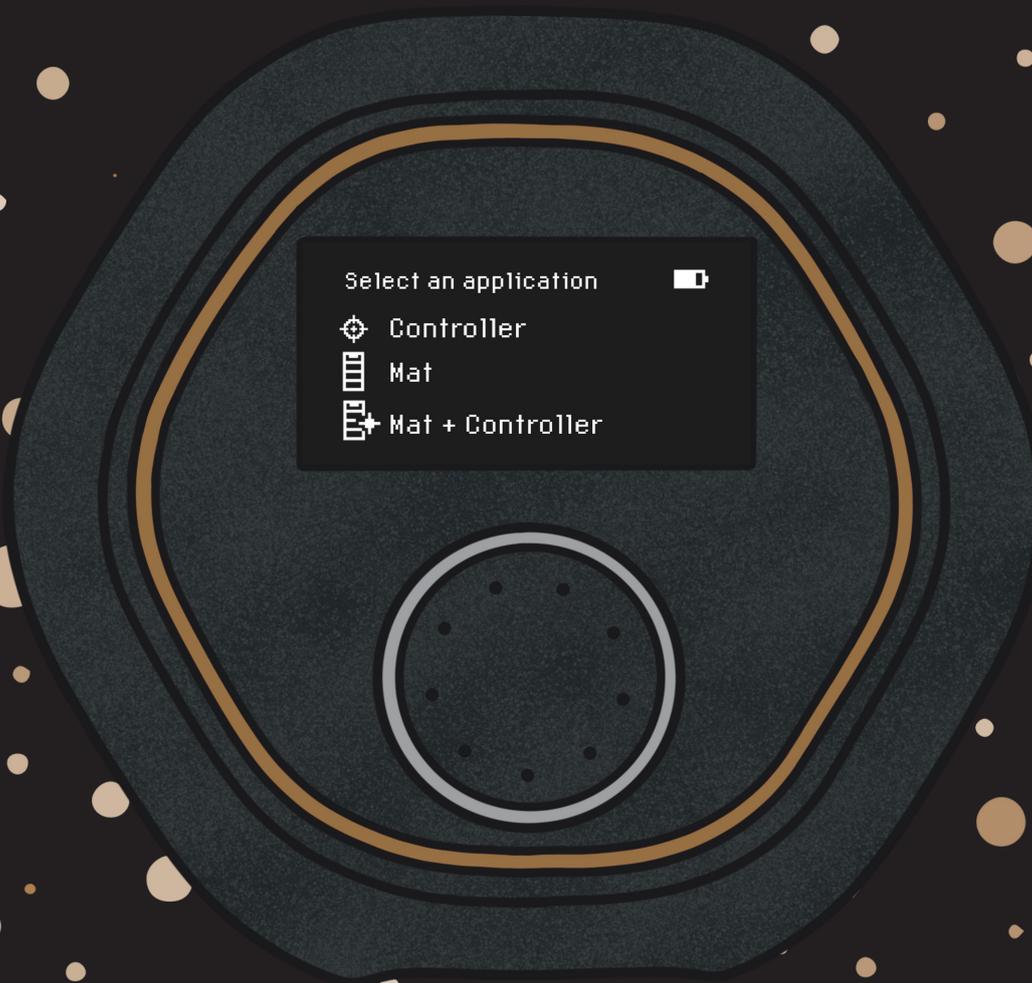

Select an application 

 Controller

 Mat

 Mat + Controller

Definitions

WARNING: Indicates a potentially hazardous situation, which, if not avoided, may result in minor or moderate injury.

CAUTION: Indicates a potentially hazardous situation, which, if not avoided, may result in damage to the device.

NOTE: Used to call attention to notable information that should be followed during use.

Contents

Definitions.....	1
Contents.....	1
Important Safety Information.....	2
Intended Use.....	4
Package Contents.....	4
Charging the CELLER8®	5
Setting Up the CELLER8®	6
Creating a Custom CELLER8® Program	13
Settings Menu	14
About CELLER8® Programs.....	17
Device Care and Maintenance	18
Troubleshooting.....	18
CELLER8® Technical Information	20
CELLER8® + CELLER8® Mat Technical Specification	21
List of CELLER8® Symbols.....	22
Warranty and Returns Information	24
Responsible Manufacturer and Contacts.....	24

Important Safety Information

WARNING: Failure to follow these instructions could result in injury, or damage to the CELLER8®, CELLER8® Mat, or other property. Read all the information below before using the CELLER8® or CELLER8® Mat.

WARNING: Do not use the CELLER8® products if fitted with a Pacemaker, ICD or any other electrical implants.

WARNING: This appliance should not be used by children under the age of 12 years.

WARNING: To prevent possible injury by entanglement, please keep cables tidy when in use.

WARNING: To prevent possible injury from poor circulation, please ensure the straps are not fastened too tightly.

WARNING: Only use the pre-approved AC charging adaptor, supplied with CELLER8® products and ensure that it is plugged into a mains location that is safe and easily accessible. Please be advised that an accessory not recommended by the manufacturer may result in injury, loss of performance and damage to the CELLER8® or CELLER8® Mat. The product warranties do not cover product failure or damage resulting from use with non-approved accessories.

WARNING: Do not open the CELLER8® or CELLER8® Mat or attempt to repair the products. Disassembling the products may damage them, and may cause injury.

WARNING: Do not use CELLER8® or CELLER8® Mat near heat sources such as open flames, radiators, heaters, heated blankets. Can cause injury and damage to device. Please refer to the technical specification for operating temperatures.

WARNING: If an adverse reaction is suspected from using the CELLER8® products, discontinue use and consult a medical professional immediately.

WARNING: Do not use the CELLER8® and/or accessories if there are visible signs of damage.

WARNING: Supervision or instruction concerning the safety of use is necessary for those classed as vulnerable persons; that includes but is not limited to persons with reduced physical, sensory, or mental capabilities.

CAUTION: Under no circumstances should the CELLER8® or CELLER8® Mat be immersed or splashed with liquid.

CAUTION: Only use CELLER8® accessories with the CELLER8® products. Please be advised that an accessory not recommended by the manufacturer may result in loss of performance and damage to the CELLER8® or CELLER8® Mat. The product warranties do not cover product failure or damage resulting from use with non-approved accessories.

CAUTION: The CELLER8® products should only be wiped down with a non-abrasive cloth. Use of cleaning techniques or products not approved by the manufacturer may result in loss of performance and damage to the CELLER8® or CELLER8® Mat.

CAUTION: Never store CELLER8® or CELLER8® Mat in a Vehicle in cold or hot weather. Please refer to the technical specification for storage temperatures and humidity ranges.

CAUTION: Never store CELLER8® for prolonged time with the battery charge depleted.

NOTE: The CELLER8® products are not medical devices and should not be used as a substitute for professional medical judgment. The CELLER8® products are not designed or intended for use the cure, mitigation, treatment, or prevention of any condition or disease. Please consult your healthcare provider prior to making any decisions related to your health.

NOTE: Ensure the CELLER8® is used within the stated operating temperature and humidity ranges. Please refer to the technical specification for operating temperatures and humidity ranges.

NOTE: During a Custom program, if the frequency is being set to 40Hz and above, ensure the CELLER8® is plugged into the mains power, as the battery alone cannot support the higher frequencies.

NOTE: Any changes to pre-set programs will not be remembered once the device has been switched off.

NOTE: A sound will not play at the end of a session running the *Night* program to reduce disturbing the user.

NOTE: If selecting an application using the CELLER8® Mat before connecting the CELLER8® Mat to the CELLER8®, the Mat Not Connected screen will appear.

Intended Use

The CELLER8® is a non-invasive Pulsed Electromagnetic Field (PEMF) device intended to be used as a localised PEMF application or to power or alongside the CELLER8® Mat for a full body PEMF application.

The CELLER8® and CELLER8® Mat deliver magnetic therapy using electromagnetic fields generated by copper coils and is used to improve overall general wellness as part of a healthy lifestyle.

CELLER8® is designed for a single user and should not be used by persons under 12 years of age.

Package Contents



The CELLER8® kit may contain the following items:

1. CELLER8®
2. CELLER8® Mat (*sold separately*)
3. AC Power Adaptor
4. Elasticated Strap
5. Quick Start Guide
6. CELLER8® Bag
7. CELLER8® Mat Bag (*included with CELLER8® Mat*)

Charging the CELLER8®


1. Charging Port
2. CELLER8® Mat Port



To charge the CELLER8®, connect the pre-approved AC Power Adaptor provided to the Charging Port (1) of the CELLER8®.



Then connect the Power Adaptor to the Mains.

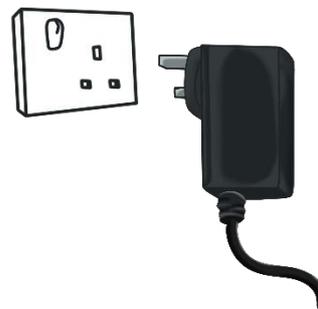


WARNING: Only use the pre-approved AC charging adaptor, supplied with CELLER8® products and ensure that it is plugged into a mains location that is safe and easily accessible. Please be advised that an accessory not recommended by the manufacturer may result in injury, loss of performance and damage to the CELLER8® or CELLER8® Mat. The product warranties do not cover product failure or damage resulting from use with non-approved accessories.

WARNING: To prevent possible injury by entanglement, please keep cables tidy when in use.


CELLER8® Switched Off

When the CELLER8® is switched off and plugged into mains power, the CELLER8® will be charging and an animated battery will appear onscreen.

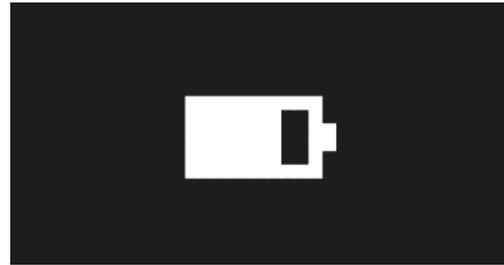

CELLER8® Switched On

When the CELLER8® is switched on and plugged into mains power, the CELLER8® will be charging and the battery status can be found in the upper, righthand corner.

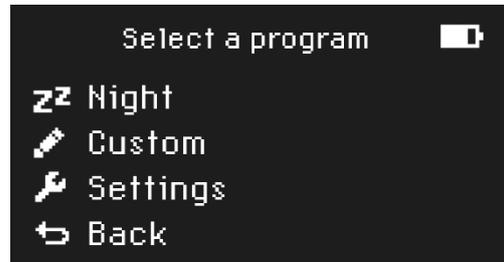


Setting Up the CELLER8®

CELLER8® Layout

1. CELLER8® Screen
2. Control Dial



CELLER8® + CELLER8® Mat Layout

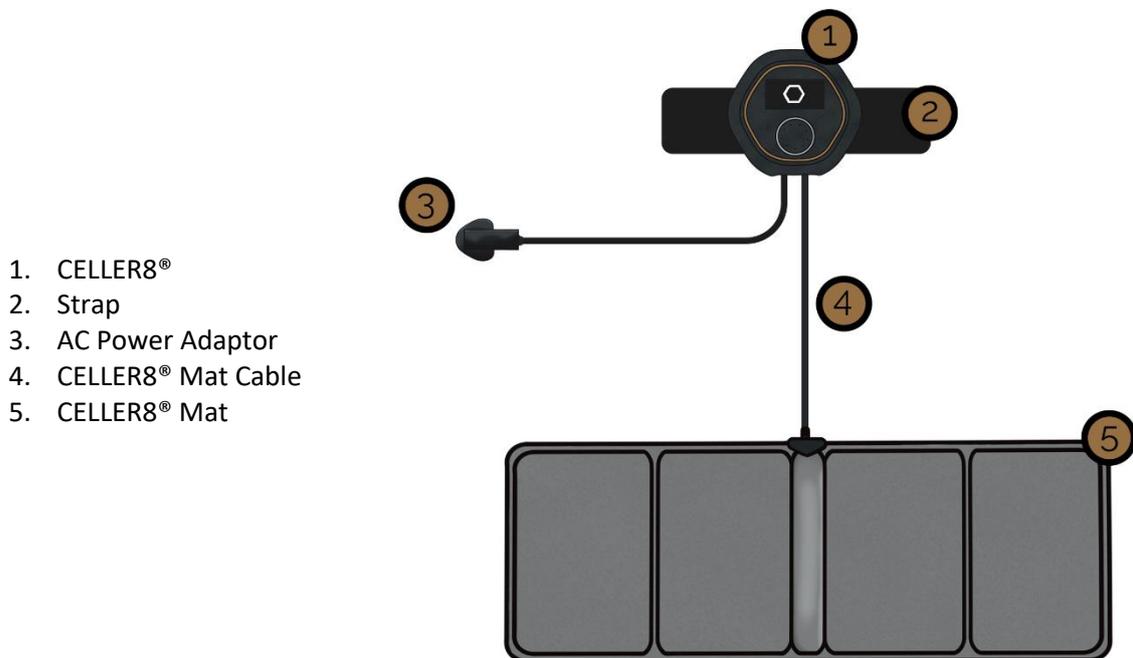

Switching On/Off the CELLER8®

To switch the CELLER8® on, push down the Control Dial for 2 seconds. An animation will appear onscreen, followed by the Menu screen, and Control Dial will illuminate to indicate that the CELLER8® is switched on.

To turn the CELLER8® off at any point, hold down the Control Dial for 3 seconds until the lights and screen turn off to indicate the device is now switched off.


The device also has an auto switch-off feature. If the device is on the Menu screen and there has been no activity for 60 seconds, it will automatically switch off. The device will also switch off automatically at the end of a program.

Navigating the CELLER8®

The CELLER8® has a Control Dial (2), which enables a quick and easy navigation of the CELLER8®. To navigate the CELLER8® menus, The Control Dial can be turned clockwise and anti-clockwise to move up and down, respectively. The Control Dial is also the button needed to make and confirm selections. To use the button, press down on the centre of the Control Dial, an audible click will be heard.

Setting up the Strap

The CELLER8® comes with a dedicated, elasticated strap, to help keep the CELLER8® in place when being used for localised application.

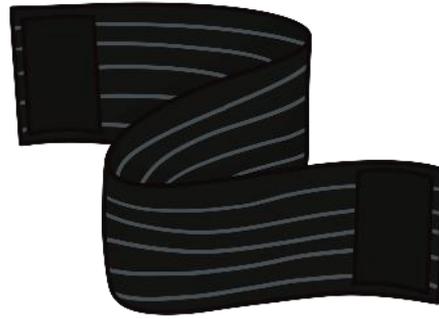


To set-up, slide the Strap through the clip on the back of the CELLER8®.

Holding the CELLER8® in place, wrap the elasticated strap around the area of the body. Secure the strap by using the Velcro. The CELLER8® and Strap should be snug but comfortable.

WARNING: To prevent possible injury from poor circulation, please ensure the straps are not fastened too tightly.


CELLER8® and Strap Examples

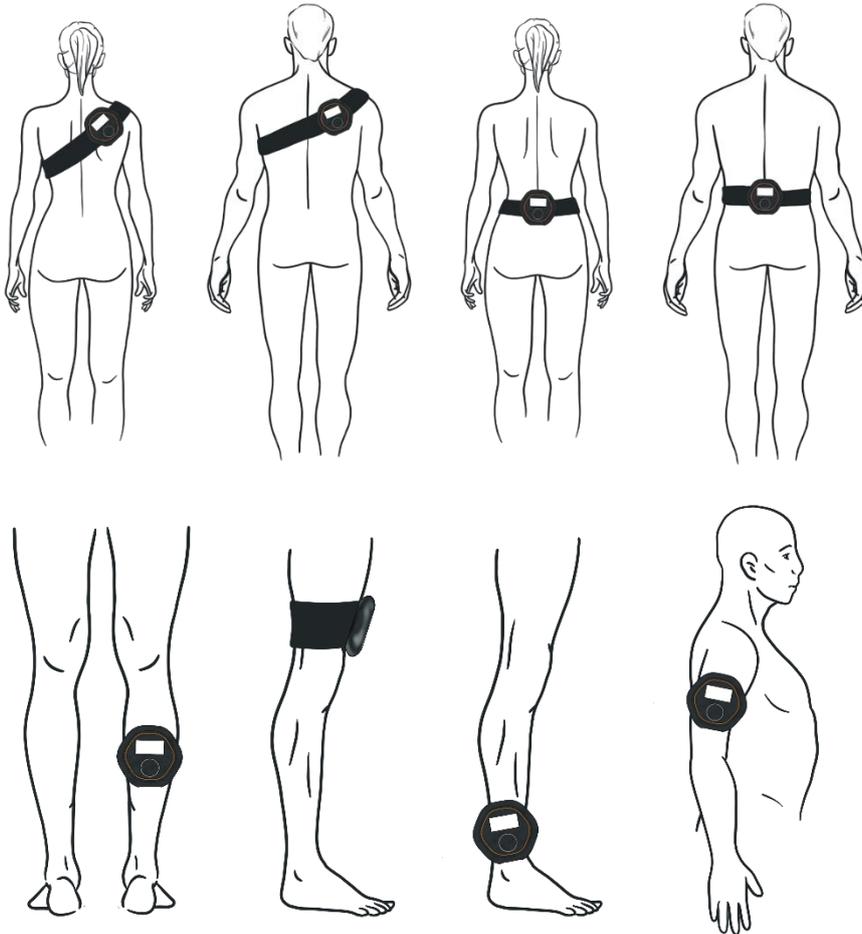

Setting up the CELLER8® Mat

The CELLER8® can be used alongside the CELLER8® Mat for full-body PEMF application.

The CELLER8® can be used as the controller for the CELLER8® Mat, or it can be used at the same time for both a full-body and localised PEMF application, as shown below.

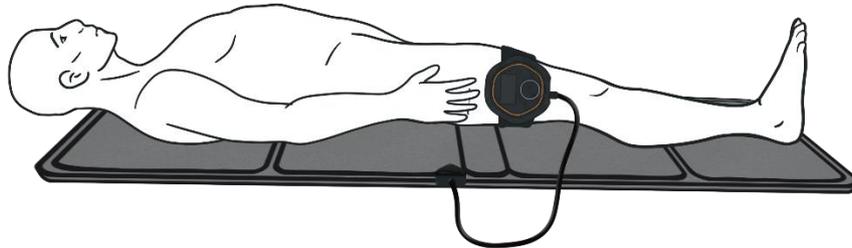

The CELLER8® can be used with the CELLER8® Mat on both battery and mains power. When the CELLER8® Mat is in use, it should be fully unfolded.

NOTE: During a Custom program, if the frequency is being set to 40Hz and above, ensure the CELLER8® is plugged into the mains power, as the battery alone cannot support the higher frequencies.

WARNING: Only use the pre-approved AC charging adaptor, supplied with CELLER8® products and ensure that it is plugged into a mains location that is safe and easily accessible. Please be advised that an accessory not recommended by the manufacturer may result in injury, loss of performance and damage to the CELLER8® or CELLER8® Mat. The product warranties do not cover product failure or damage resulting from use with non-approved accessories.

WARNING: To prevent possible injury by entanglement, please keep cables tidy when in use.

To use the CELLER8® Mat with the CELLER8®, plug the CELLER8® Mat Cable into the CELLER8® Mat Port (2).




Selecting a CELLER8® Application

When the CELLER8® is switched on, the screen will display the Application Menu. There are three application options:


CELLER8® (Controller)

The CELLER8® application will apply localised PEMF.

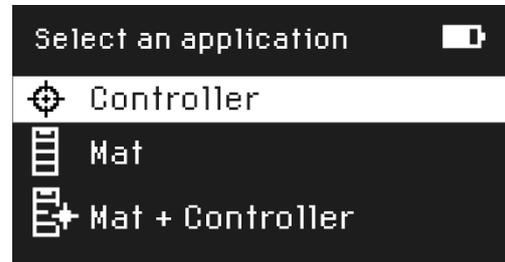

CELLER8® Mat

The CELLER8® Mat application will only use the CELLER8® Mat to apply full-body PEMFs and must be plugged into the CELLER8® in order to be used. Only the CELLER8® Mat will emit PEMFs in this mode. Please ensure the CELLER8® Mat is connected to the CELLER8® before selecting this option.

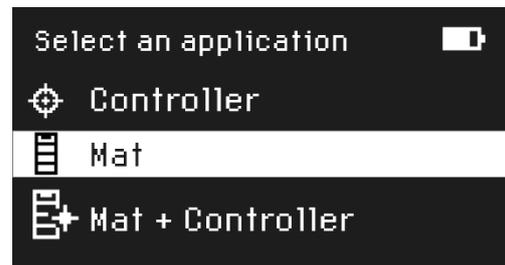

CELLER8® Mat + CELLER8® (Controller)

The CELLER8® Mat + CELLER8® application enables the CELLER8® Mat to apply PEMF to the entire body but also uses the CELLER8® to apply localised PEMF at the same time. Both the CELLER8® Mat and the CELLER8® will emit PEMFs in this mode. Please ensure the CELLER8® Mat is connected to the CELLER8® before selecting this option.



NOTE: If selecting an application using the CELLER8® Mat before connecting the CELLER8® Mat to the CELLER8®, the Mat Not Connected screen will appear.

If this error persists, please refer to the Troubleshooting section in this manual.




Select the desired application using the Control Dial and press the Control Dial to select.


Selecting a CELLER8® Program

Once the desired applicator has been selected, the Program Menu will appear. There are 4 pre-set programs to choose from:

- Morning
- Evening
- Meditation
- Night

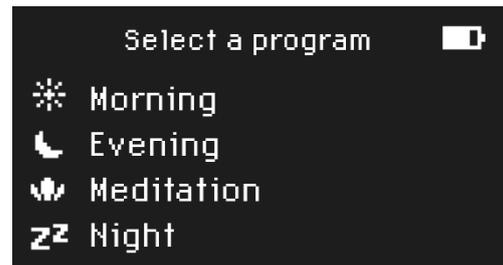

Alternatively, you can opt for the Custom program. More information about the programs can be found in the About CELLER8® Programs section of this manual.

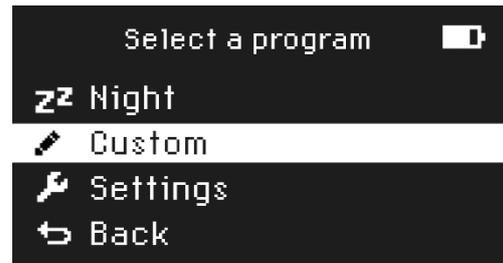

To return to the Application Menu, use the Control Dial to scroll to *Back* and press the Control Dial to return.

Selecting Intensity and Time


Once one of the pre-set programs have been selected, the *Intensity* (strength of the magnetic field) and *Time* (duration of the session) can be modified.

The pre-set programs come with a recommended *Intensity* and *Time* already pre-loaded. These will be prompted every time the device is used. These can be changed if desired.

To change the pre-set intensity, use the Control Dial to highlight *Intensity*. Press the Control Dial to open the adjustment screen.



Turn the Control Dial to adjust the Intensity as desired. Press the Control Dial to confirm.

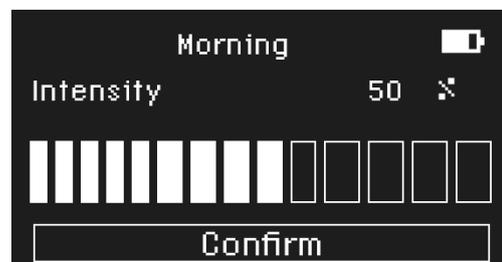

Setting	CELLER8® Mat	CELLER8®
2%	1 Gauss/ 100 μ T	2 Gauss/ 200 μ T
4%	2 Gauss/ 200 μ T	4 Gauss/ 400 μ T
10%	5 Gauss/ 500 μ T	10 Gauss/ 1,000 μ T
20%	10 Gauss/ 1,000 μ T	20 Gauss/ 2,000 μ T
40%	20 Gauss/ 2,000 μ T	40 Gauss/ 4,000 μ T
60%	30 Gauss/ 3,000 μ T	60 Gauss/ 6,000 μ T
80%	40 Gauss/ 4,000 μ T	80 Gauss/ 8,000 μ T
100%	50 Gauss/ 5,000 μ T	100 Gauss/ 10,000 μ T

The localised CELLER8® will reach intensities up to 100 Gauss (10,000 Microtesla).

The full body CELLER8® Mat will reach intensities up to 50 Gauss (5,000 Microtesla)

To change the time, use the Control Dial to highlight *Time*. Press the Control Dial to open the adjustment screen.



Turn the Control Dial to adjust the Time as desired. Press the Control Dial to confirm.



If the pre-set *Intensity* and/or *Time* are adjusted, this screen will appear showing the new values.

To begin the program, ensure *Start* is highlighted using the Control Dial and press the Control Dial to start the program.



To make any changes, highlight the *Back* button and press the Control Dial to go back a step.

NOTE: Any changes to pre-set programs will not be remembered once the device has been switched off.



Once the program has started, information about the running program will appear onscreen.

The selected program will appear at the top of the screen and the timer will begin to count down. On the left of the screen, an animation of a body will appear to recharge.



This screen will dim after 15 seconds as a power saving feature.

To pause the program, press the Control Dial; press once if the screen is active, or twice if the screen has dimmed. A prompt will appear giving the option to Stop or Resume the program. If the program is to be resumed after a break, use the Control Dial to ensure Resume is highlighted and Control Dial to resume the current session. To end the program early, ensure Stop is highlighted and press the Control Dial to end the current session and the device will return to the menu screen.

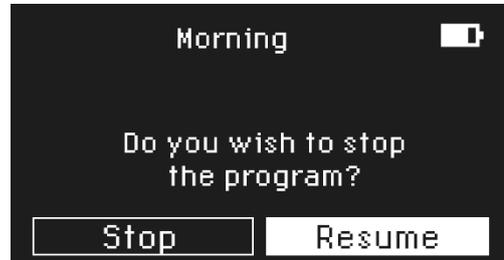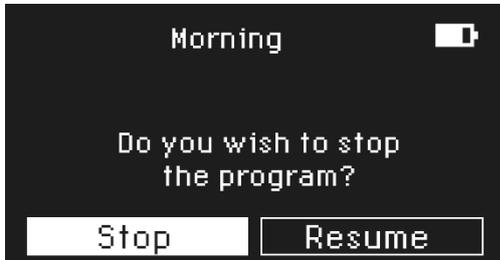

Once a PEMF session has ended, a short animation will bring the session to a close. If the sound is activated in the settings, a buzzer will also sound. The device will then auto power off.


NOTE: A sound will not play at the end of a session running the *Night* program to reduce disturbing the user.

Creating a Custom CELLER8® Program

Creating a *Custom* program similar to changing the parameters in the pre-set programs, however, the custom program will only run one frequency that has been selected for the entire session.

NOTE: During a *Custom* program, if the frequency is being set to 40Hz and above, ensure the CELLER8® is plugged into the mains power, as the battery alone cannot support the higher frequencies.

After an application mode has been selected from the Application Menu, select the *Custom* program from the Program Menu.

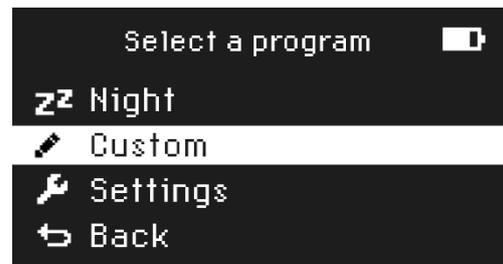

With the *Custom* program, the *Intensity*, *Time*, and *Frequency* can be customised.

To change a parameter, use the Control Dial highlight it, then the Control Dial to open the adjustment screen. Turn the Control Dial to adjust the Time as desired. Press the Control Dial to confirm.






Once the parameters for the *Custom* program have been made and confirmed, a screen will appear showing the new values.

NOTE: Any changes to the *Custom* program will be remembered once the device has been switched off in the memory as the last used *Custom* program.

As with the pre-set programs, information about the running program will appear onscreen.

The selected program will appear at the top of the screen and the timer will begin to count down. On the left of the screen, an animation of a body will appear to recharge. This screen will dim after 15 seconds as a power saving feature.

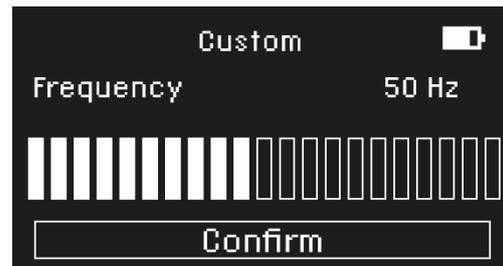


Once a PEMF session has ended, a short animation will bring the session to a close. If the sound is activated in the settings, a buzzer will also sound. The device will then auto power off.

Settings Menu

In the Program Menu, use the Control Dial to navigate to the *Settings*.

Press on the Control Dial to open the Settings Menu.

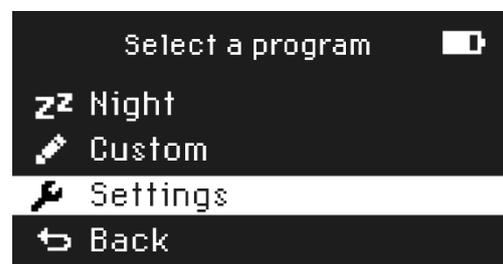

In the Settings Menu, Sound can be enabled/disabled and the Polarity of the CELLER8® products can be altered.

The Settings Menu also displays the current firmware version.



Enabling/Disabling Sound

Use the Control Dial to highlight *Sound*.

Press the Control Dial to enable/disable sound.

When sound is disabled, no sound will play at the end of a PEMF session to signify the user.



Use the Control Dial to navigate to *Confirm* and press the Control Dial to save the changes.

These changes will be remembered.



Changing Polarity

Changing the Polarity of the magnetic fields in the Settings Menu will change the direction of the magnetic field around the mat.

As standard, the back of the CELLER8® is set to North.



If using the CELLER8® Mat, the grey side of the CELLER8® Mat is set to North as standard.



To change the Polarity, in the Setting Menu, use the Control Dial to highlight *Polarity*.

Press the Control Dial to change the Polarity.




Use the Control Dial to navigate to *Confirm* and press the Control Dial to save the changes.


These changes will be remembered.



When the Polarity is switched to South, the face of the CELLER8® will be North.

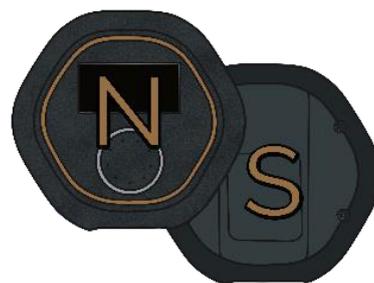

When the Polarity is switched to South, the grey side of the CELLER8® Mat will be South.

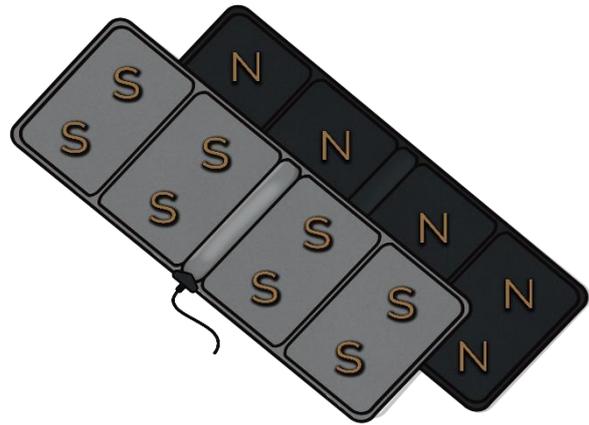

About CELLER8® Programs

The CELLER8® has 4 pre-set programs to choose from:

- Morning
- Evening
- Meditation
- Night

Alternatively, you can opt for the Custom program.

Morning Program (12Hz to 30Hz, stepping up every 5 seconds)

The Morning Program uses a range of frequencies specifically for application at the earlier part of the day. The frequency will step up by 1Hz every 5 seconds, which will apply a good range of different frequencies in one session.

The Morning Program has been designed to help with alertness, concentration, focus, recovery.

Evening Program (8Hz & 10Hz, every minute)

The Evening Program runs 8Hz and 10 Hz, alternating every minute.

The Evening Program has been designed for relaxation, recovery, or as a natural wellness program.

Meditation Program (from 8Hz to 4Hz, stepping down every 30 seconds)

The Meditation Program runs from 8Hz, and every 30 seconds steps down until it reaches 4Hz. After 30 seconds the frequency will return to 8Hz and the process will repeat until the end of the session.

The Meditation Program has been designed to help find a very relaxed state of mind, similar to that in deep meditation.

Night Program (from 4Hz to 1Hz, over 10 minutes)

The Night Program runs from 4Hz, stepping down to 1Hz over 10 minutes. After 10 minutes the frequency will return to 4Hz and the process will repeat until the end of the session.

The Night Program has been designed to be used at bed time. In this mode, it can be used for up to 12 hours, but is set for 2 hours as default. The device lights will dim in this mode and the sound will be disabled, even if the sound is selected enabled on the CELLER8®. This is so that the device does not wake the user when the program finishes.

Custom Program (1Hz – 100Hz)

The Custom Program will only run the chosen frequency for the duration of Time/Intensity selected. The frequency range to choose from is between 1 – 100 Hz.

Device Care and Maintenance

Cleaning the CELLER8® Products

CAUTION: The CELLER8® products should only be wiped down with a non-abrasive cloth. Use of cleaning techniques or products not approved by the manufacturer may result in loss of performance and damage to the CELLER8® or CELLER8® Mat.

CAUTION: Under no circumstances should the CELLER8® or CELLER8® Mat be immersed or splashed with liquid.

Storing the CELLER8® Products

Always store CELLER8® in a cool, dry place.

CAUTION: Never store CELLER8® or CELLER8® Mat in a Vehicle in cold or hot weather. Please refer to the technical specification for storage temperatures and humidity ranges.

CAUTION: Never store CELLER8® for prolonged time with the battery charge depleted.

Troubleshooting

Programs Not Running as Expected

If a problem is encountered in the operation of the CELLER8®, first turn the device off, and then back on again. This will allow the programming to reboot and reset itself. Next, ensure the device is sufficiently charged.

If the problem persists, please contact the CELLER8® Customer Care Team.

Warning Screens

Battery Charging

When the CELLER8® is switched off and plugged into mains power, an animated battery will display onscreen to signify that the battery is charging.



Batter Low

The Battery Low screen will show when the device does not have enough battery to operate. In this instance, the device must be charged to a sufficient amount before it can be used.



If this problem persists, please contact the CELLER8® Customer Care Team.

Mat Not Detected

The Mat Not Detected screen will show when an application using the CELLER8® Mat has been selected before the CELLER8® Mat has been connected. Please ensure the CELLER8® Mat is connected for selecting the application.



This screen may also show if the connection is not secure. Please unplug the CELLER8® Mat and reconnect it.


If the CELLER8® Mat is still not detected, please try turning the CELLER8® off and on again to reset the programming.

If this problem persists, please contact the CELLER8® Customer Care Team.

CELLER8® Technical Information

CELLER8® and CELLER8® Mat Coil Locations

The CELLER8® and CELLER8® Mat consist of copper coils which generate a magnetic field. A frequency generator is used to energize the coil to create a pulsed electromagnetic field. One large copper coil can be found in the CELLER8® and 12 copper coils can be found in the CELLER8® mat to give the best coverage of the magnetic fields.

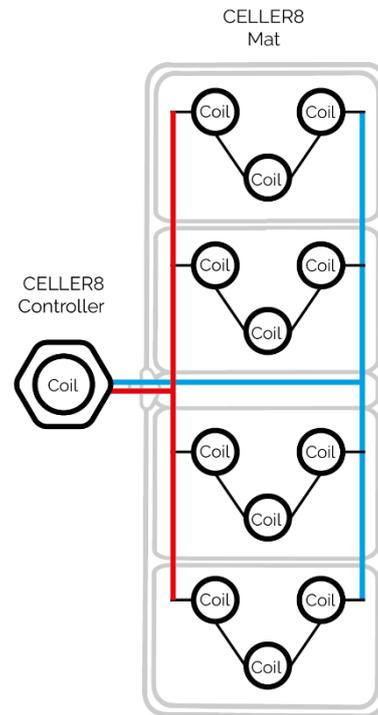

CELLER8® + CELLER8® Mat Technical Specification

Intensity Range	1G-100G (Gauss) 0.1mT-10mT (Milli-Tesla) 100µT-10,000µT (Micro-Tesla)
Frequency Range	1-100Hz
Programs	5 (including <i>Custom</i>)
Application	CELLER8® Mat, CELLER8®, and CELLER8® + CELLER8® Mat
Number of Coils	CELLER8®: 1 CELLER8® Mat: 12
Polarity	North and South (user selectable)
Waveform	Square Pulsed
Warranty	CELLER8®: 3 years CELLER8® Mat: 6 months
Power Source	Battery operated and Mains
Dimensions	CELLER8®: 14cm x 14cm x 4cm CELLER8® Mat: 49cm x 173cm x 2cm
Models and Part Number	CELLER8
Operation Time	100 minutes when running <i>Morning</i> program 36 hours when running <i>Night</i> program <small>(When operating with full battery and running the program on both the CELLER8® and CELLER8® Mat at their default intensity).</small>
Operator	End User
Operating Temperature	0°C-50°C (32°F-122°F)
Operating Humidity	10-95% RH (non-condensing)
Storage and Transportation Temperature	0-50°C (32°F-122°F)
Storage and Transportation Humidity	10-95% RH (non-condensing)
Weight	CELLER8®: 240g Power Supply: 160g CELLER8® Mat: 3.75kg
Rechargeable Battery	RRC1120 Li-ion 3.6v 2.35aH
External Power Supply	XP Power VEP24US24 Input: 100-240v – 0.6A, 50/60Hz Output: 24v – 1.0A

CELLER8® Strap Technical Specification

Materials	Nylon Multi_lament 25% Nylon Mono_lament 50% Elastane 25%
Dimensions	100cm x 7.5cm
Weight	65g

List of CELLER8® Symbols

Icon	Description	Icon	Description
	Back		CELLER8® Program
	Battery		Battery Full
	Battery Empty		Battery Empty
	CELLER8® hexagon		Meditation Program
	Custom Program		Morning Program
	Evening Program		Night Program
	CELLER8® Mat + CELLER8® Program		CELLER8® Mat Program
	No Mat Detected		Percentage
	North Polarity		South Polarity
	Program in progress		Program Complete
	Settings		

List of Packaging Symbols

Symbol	Description
	Manufactured by
IPX0 – not protected against water ingress	IPX0 – not protected against water ingress
	Direct current
	CE mark
	Keep dry
	Consult instructions before use
	Serial Number
	UDI number
	Dispose of according to WEEE
	Class III appliance for use with a separate extra-low voltage external power supply
	Manufacturer logo
	Distributed by logo
	General symbol for recovery/recyclable
	UKCA
	Humidity limitation
	Temperature limit

Warranty and Returns Information

The CELLER8® is warranted to be free from manufacturer defect in materials and workmanship for a period of 3 years for the CELLER8® and 6 months for the CELLER8® Mat, from the date of purchase, subject to normal use of the device. This warranty only extends to the original device purchaser so long as purchased directly through CELLER8® or an authorized CELLER8® affiliate, distributor, or retailer. If the product is believed to be defective within the warranty period, then contact the CELLER8® Customer Care Team.

Alternatively contact the local authorised distributor or retailer the CELLER8® was bought from.

CELLER8® reserves the right to determine if a product is defective and eligible for warranty replacement, and to replace a defective product with the most comparable product currently available. Proof of valid purchase is required for warranty eligibility. Please review the terms of service and detailed warranty information on the CELLER8® website, at www.celler8.com.

Please contact the authorised retailer from where the CELLER8® device was bought from to obtain a valid return address. Devices returned to the manufacturers' address may get lost and unassigned from a customer so please contact the retailer from where the device was bought from in the first instance to obtain the correct return address.



Dispose the CELLER8® in accordance with WEEE guidelines.
Never dispose of any electronic instrument or batteries in domestic waste.
At the end of the product's life, contact the CELLER8® Customer Care Team or its distributor for disposal instructions.

Responsible Manufacturer and Contacts

Contact the local distributor for support and repairs for the CELLER8®.

Contact Details:

Phone: +44 (0) 1622 854864

Email: office@celler8.com

Website: www.celler8.com

Distributed by: NewMed Ltd
Station Yard, Station Road,
Harrietsham,
Maidstone, Kent,
ME17 1JA
United Kingdom
www.newmedltd.co.uk

Manufactured by Bedfont® Scientific Ltd.
Station Yard, Station Road,
Harrietsham,
Maidstone, Kent,
ME17 1JA
United Kingdom
www.bedfont.com
ask@bedfont.com

info@newmedltd.co.uk
0808 1689 282

0044 (0)1622 851122


Disclaimer: The information contained in this manual is for informational and educational purposes only and come from our own experiences. The product does not claim to diagnose, treat, cure or prevent any medical conditions. Always consult your doctor.



Email: office@celler8.com
Phone: +44 (0) 1622 854864



Bedfont® Scientific Ltd.
Station Road, Harrietsham,
Maidstone, Kent,
ME17 1JA England

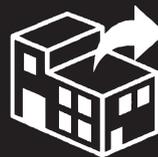

NewMed Ltd.
Station Road, Harrietsham,
Maidstone, Kent
ME17 1JA England
Tel: +44 (0) 1622 854864.
Email: office@celler8.com
www.celler8.com

© Bedfont® Scientific Limited 2023

Issue 1 - June 2023, Part No: LAB875
Bedfont® Scientific Limited reserves the right to change or update the literature without prior notice.
Registered in: England and Wales. Registered No: 1289798

